Silla de ruedas activa de aluminio
Active Aluminum Wheelchair
Cadeira de rodas ativa de alumínio

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SPECS

Material	Rigid aluminum, Black liquid paint
Seat Width	40.6 cm & 46cm 16" & 18"
Seat Depth	40.6 cm & 46cm 16"
Backrest Height	14" & 16"
Weight Capacity	110 kg.
Product Weight	14 kg.
Rear Wheels	24" Pneumatic, quick release, anodized ring
Front Wheels	5" x 1.5" solid Black aluminum rim
Front Fork	Aluminum
Seat Height	Adjustable
Gravity Center	Adjustable
Backrest Angle	Adjustable
Foot Support	Fold-down and adjustable aluminum platform
Total Height	80 cm 31.5"
Overall Depth	83.5 cm 32.5"
Total Width	63 cm 30"

USAGE

Foot Support

The foot support can be folded up so that the user can do front transfers.


Seat Height Adjustment

To adjust the rear seat height, loosen and remove the 2 screws (1 on each side) that secure the wheel mounting plate to the chair frame. Move the screws to the desired height and retighten the cap screws.

Note: Rear seat height adjustments may also require front wheel angle adjustment.

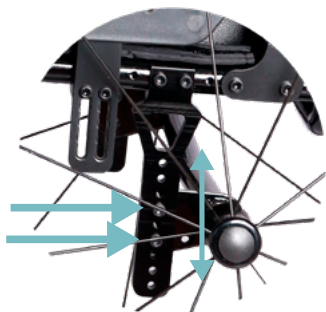
Center of Gravity Adjustment

To Move and Adjust the Center of Gravity

If the center of gravity is moved to the front, the less force will be required to turn the chair. The further back the shaft and clamp are, the more stable the Active Aluminum Wheelchair will be.

Slide the wheel mounting plate forward or backward as needed.

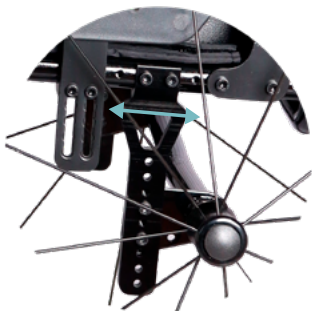
Note: Changing the center of gravity reduces the risk of a fall or rollover. Make sure the chair does not tip back when you lean.

To Move and Adjust the Center of Gravity

Loosen the socket head screws. Slide the camber tube assembly forward or rearward along with the camber tube. Retighten the screws. Be sure to measure the new position for uniformity on both sides of the Active Aluminum Wheelchair.

After securing the axle plate clamps, install the rear wheels, sit in the chair and maneuver while another person serves as a spotter to see if the new fit is right for you.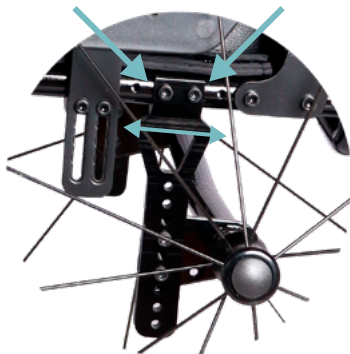

Backrest Angle Adjustment

1. Release the backrest by pulling the adjustable tension cord in a short, easy motion.
2. While maintaining tension on the release cord, rotate the backrest to the desired angle, forward or backward.
3. Secure the backrest in place by releasing the cord and allowing the pins to enter the nearest hole in the backrest clamp.

Folding and Unfolding of the Backrest

1. Release the backrest by pulling on the adjustable tension cord in a short, easy motion.
2. While maintaining tension on the release cord, rotate the backrest until it is deployed. Remove the cushion and push down firmly until it locks in the folded position.
3. Lock the backrest in the folded position when removing the cushion. Release the cord and push down allowing the pins to enter the folding position hole of the backrest clamp.

Side Clothing Protector Adjustment

Turn the two hollow screws until the mounting bar is released. Slide the clothing protector to the desired height and then tighten the screws until secure. Check that they are fixed by applying pressure on them.

Anti-tips

To install or remove, press the pin, turn, and install or remove it. They can also be installed in reverse to maneuver the chair.

SAFETY

Stability

The Active Aluminum Wheelchair Coyote A is designed to be safe and stable as long as center of gravity is maintained while going about normal activities. Care should be taken when performing activities that cause the user's center of gravity to shift on the aluminum active wheelchair.

Never reach so far that you need to shift in the seat or bend over out of the Active Aluminum Wheelchair.

When you reaching for objects that require you to lean, be certain front casters are pointed in a forward position that extends the wheel base and stabilizes the Active Aluminum Wheelchair.

Ramps

Be sure of your own capabilities, limitations and strength before attempting to go up a ramp. Correct techniques on ramp navigation should be shown to you by a qualified professional.

If you must stop on an incline, avoid sudden, abrupt movement when you resume movement. Backward angling of the Active Aluminum Wheelchair or sudden abrupt movement could cause instability.

Attempting to use the wheel lock brakes could result in accidental locking, tipping to one side, sudden stopping.

Avoid changing direction when going down an incline, as this could result in instability or tipping over.

Never use your Active Aluminum Wheelchair Coyote A on an incline steeper than 4.8°.

WARRANTY

Loh Medical offers a 12 month warranty on the frame of the Active Aluminum Wheelchair Coyote A.

Any defective frame components will be replaced at no cost.

Please contact your local Loh Medical sales rep for warranty service. For more information on your local Loh Medical sales rep, please visit:

<https://www.lohmedical.com/en/contact>

The Active Aluminum Wheelchair Coyote A has been tested and certified in accordance with CE requirements and BSEN 12183.

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