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ENGLISH**INDEX**

1. Product Introduction	16
1.1 Product Information	16
2. Maintenance and Repair	16
2.1 Recommendations	16
2.2 Disinfecting	17
2.3 Tire Maintenance	17
2.4 Intended Use	17
3. Refurbishing, Tilt, and Bend	17
3.1 Reaching, Leaning, and Forward Tilt	17
3.2 Reaching, Leaning, and Backward Bend	17
3.3 Inclines/Stairs	17
3.4 Stairs	18
4. Parts	18
4.1 Lateral Supports	18
4.2 Armrests	18
4.3 Abductor	19
4.4 Detachable Footrest	19
4.5 Raise/Lower Footrest	20
4.6 Leg Support Height Adjustment	20
4.7 Head Support	21
5. Basic Operation	21
5.1 Unfolding and Folding the Chair	21
5.2 Unfolding	21
5.3 Folding	21
6. General Recommendations	22
6.1 Stability	22
6.2 Safety Information	22
6.3 Locking Device	23
6.4 Anti-Tip	24
6.5 Tilting and Reclining	25
6.6 Tray (Optional)	25
7. Warranty	26

1. PRODUCT INTRODUCTION

1.1 Product Information

Read the user manual carefully before using the product.

Frame	Blue and red aluminum folding chair
Seat Width	14" and 16"
Seat Depth	14"
Product Weight	21 kg
Weight Capacity	90 kg / 198.4 lbs
Front Wheels	6" solid
Rear Wheels	16" solid
Seat, Backrest	Breathable padding
Armrests	Height adjustable
Footrest	Raise and detachable
Headrest	Padding
Backrest Height	18", 22" with Headrest
Tray	Optional

2. MAINTENANCE AND REPAIR

2.1 Recommendations

1. Store the **Coyote T** in a dry place and avoid caustic chemicals such as oil, acid, alkali, etc.
2. Keep the **Coyote T** clean. Use water and soap to clean it. Dry it quickly and apply a little lubricating oil to the moving parts after cleaning.
3. Avoid use in rain and do not submerge in water. Do not shower with the **Coyote T**. If the **Coyote T** gets wet, dry it quickly and apply a little lubricating oil to the moving parts.
4. This is a portable wheelchair for daily life. Do not engage in sports activities in the **Coyote T**.
5. Regularly check tire air pressure and wear on the rear tires. Replace if necessary.
6. The driving wheel spokes should be adjusted once a year.
7. Add two or three drops of high-quality motor oil to the drive wheels and front wheels each month.
8. Clean the wheel bearing of hair and other debris frequently.
9. For safety, regularly check the brake and backrest operation.

2.2 Disinfecting

Please perform any disinfection under the guidance of a technician only.

2.3 Tire Maintenance

The rear wheels of the Coyote T are solid.

Regularly check tire wear. Replace if necessary.

Adjust the wheel brake regularly to match tire wear.

2.4 Intended Use

The **Coyote T** is designed for pediatric users with disabilities. It is a manual wheelchair.

3. REFURBISHMENT, TILT, AND BEND

3.1 Reaching, Leaning, and Forward Tilt

Position the front wheels so they extend as far forward as possible and engage the wheel brakes.

Do not attempt to reach objects if you have to move forward in the seat or lift them from the floor by reaching down between your knees.

3.2 Reaching, Leaning, and Backward Bend

Position the wheelchair as close to the object as possible and the front wheels as frontal as possible, move back only with an extended arm maintaining the seated position.

3.3 Inclines/Stairs

When tilting the **Coyote T**, it requires two supports, the second assistant must be positioned on the front of the non-detachable, part of the wheelchair frame, by lifting it up and stabilizing it, when lowering to the ground.

The first assistant should stand on the curb and turn the **Coyote T** so that the rear wheels are against the curb.

Turn the anti-tips so that the anti-tip wheels are back up, the **Coyote T** should tilt backward to the balance point and, in a continuous upward movement, the rear wheels should be pulled up and over the curb.

3.4 Stairs

1. Remove the anti-tips from the **Coyote T**.
2. Similar steps with the above recommendations.
3. The wheelchair should not be lowered until the last step is clear and the **Coyote T** has moved away from the stairs.

4. PARTS

4.1 Lateral Supports

Used to prevent users from losing balance, see Image 1.

Adjustment: Press the red button on the latch to release the support, then pull the trunk support to adjust as needed and re-lock the latch. There are two possible heights for this support on the backrest plate, which operate in the same way.

Image 1

4.2 Armrests

Used to secure the seated position of users, see Image 2.

Operate the adjustment knob to install, move the armrest, or adjust its height. After any adjustment, ensure the knobs are secure.

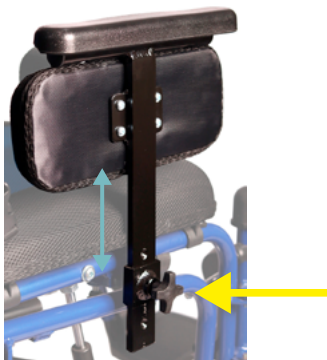
Image 2

4.3 Abductor

 **NOTE:** For this procedure, see Image 3.

The abductor is used to prevent the user from exiting due to inertia.

To install or remove the abductor, use the adjustment knob.

Image 3

4.4 Detachable Footrest

Detaching: Pull the black pin and then rotate the footrest outward. Pull the footrest until it clears the pin.

Installation: Place the pin in the hole from one side, then swing the footrest until it is locked in place on the wheelchair frame, see Image 4.

Image 4

4.5 Raise/Lower Footrest

 For this procedure, see Image 5.

Raising: Pull the footrest vertically to the desired position.

Lowering: Hold the footrest and push the control handle toward the rear wheel of the **Coyote T** at the same time. The footrest will automatically unfold. Release the controller after finishing the adjustment.

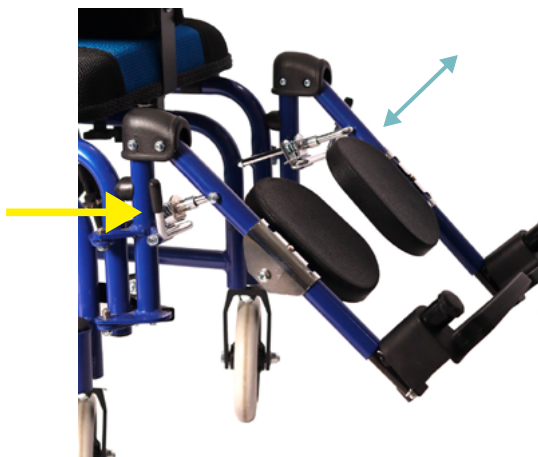
Image 5

4.6 Leg Support Height Adjustment

There are three options for the leg support height. Adjust the support until it can move, to the correct position and confirm that the button is locked in place, see Image 6.

Image 6

4.7 Head Support

Turn the knob to release the support and it can be removed vertically from the wheelchair frame.

When installing, place the support on the wheelchair frame and ensure that the lock button is in the receiver hole. To adjust the support height, operate the height adjustment knob.

5. BASIC OPERATION

5.1 Unfolding and Folding the Chair

NOTE: Remove the seat, backrest, and cushion before folding or unfolding the **Coyote T**!

To separate, press the button. The pin will come out of the wheelchair frame, then you can remove the seat and backrest. When installing, make sure the pins are securely locked in the wheelchair frame.

5.2 Unfolding

Press the vertical support bar down until the **Coyote T** is fully unfolded. Install the seat, backrest, and pillow, then unfold the footrests.

5.3 Folding

Turn the footplates upward to the vertical position, remove the seat, backrest, and pillow, then lift the vertical support bar to fold the **Coyote T**.

6. GENERAL RECOMMENDATIONS

6.1 Stability

Before using any tilt position of the **Coyote T**, ensure that the rear wheels are in the best position to maintain wheelchair stability.

The seat height, seat angle, seat frame pivot point, seat system, wheel position, rear wheel size and position, as well as the user's condition, are directly related to the stability of the **Coyote T**.

Any change in one or any combination above can cause the **Coyote T** to lose stability. These adjustments must be made by an authorized dealer or qualified technician.

Before using the tilting option, make sure the anti-tip wheel sets are in the lowest adjustment hole. (Closest to the ground/floor adjustment hole).

6.2 Safety Information

1. Unless otherwise stated, all service and adjustment must be performed while the **Coyote T** is covered.
2. Always keep hands and fingers away from moving parts to avoid injury.
3. DO NOT attempt to climb or descend a slope with a film of water, ice, or oil.
4. DO NOT operate on roads, streets, or highways.
5. DO NOT attempt to pass over curbs or obstacles. Doing so may cause your **Coyote T** to overturn and cause bodily harm or damage to the **Coyote T**.
5. DO NOT attempt to reach objects if you have to move forward in the seat.
6. DO NOT attempt to reach objects if you have to lift them from the floor by stretching between your knees.
7. DO NOT lean over the top of the backrest upholstery to reach objects behind you, as this may cause the **Coyote T** to overturn.
8. DO NOT shift your weight or sitting position towards the direction you are reaching, as the **Coyote T** may overturn.
9. DO NOT attempt to stop a moving **Coyote T** with wheel locks. Wheel locks are not brakes.

10. DO NOT tilt the **Coyote T** without assistance.
11. DO NOT use an escalator to move a **Coyote T** between floors. Serious bodily injury may occur.
12. DO NOT attempt to lift the **Coyote T** by any removable (detachable) parts. Lifting through any removable (detachable) part of the **Coyote T** may cause injury to the user or damage to the **Coyote T**.
13. DO NOT stand on the frame of the **Coyote T**.
14. DO NOT use the footrest as a platform. When entering or exiting the **Coyote T**, ensure the footplates are in the up position.

6.3 Wheel Locking Device

The wheel lock is not a brake, DO NOT use the wheel lock to slow down or stop the **Coyote T**.

The wheel lock is used to prevent movement while stationary, see Image 7.

Image 7

Pull the wheel lock handle until the rear wheel cannot move, place the automatic wardrobe in a locking position, when the **Coyote T** is automatically locked.

Pull up again on the wheel lock handle to release it, see Image 8.

Image 8

6.4 Anti-Tip

NOTE: Anti-tips should maintain a clearance of 1 1/2 to 2 inches between the bottom of the anti-tip and the floor.

NOTE: For this procedure, see Image 9.

Operate the bolt to install, remove, or adjust the anti-tip.

NOTE: Anti-tips should be fully engaged and the bolt protruding completely from the adjustment holes. Please review before use.

Image 9

6.5 Tilt & Reclining

The gas piston controls the tilting of the **Coyote T**. There are two handles to control two gas pistons, **the left one is for seat tilting and the right one is for backrest reclining**.

Pull up on the adjustment handle to engage the gas piston, release the handle when the seat reaches the angle you desire.

The angle range between the seat and the floor is 0° - 30° , and between the backrest and the floor is 90° - 180° .

6.6 Tray (Optional)

The optional accessory is the table for daily activities.

To mount and dismount the table, use the hexagon wrench (crosshair) to adjust the table.

When the table is in use, push it up until it is locked in place.

7. WARRANTY

Loh Medical offers a 12 month warranty on the frame of the **Coyote T**.

Any defective frame components will be replaced at no cost.

Please contact your local Loh Medical sales rep for warranty service. For more information on your local Loh Medical sales rep, please visit:

<https://www.lohmedical.com/en/contact>

The Active Aluminum Wheelchair Coyote A has been tested and certified in accordance with CE requirements and BSEN 12183.

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