FELIX^e

Silla motorizada compacta
Compact Power Chair
Cadeira motorizada compacta

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<https://www.lohmedical.com/es/contacto>

ENGLISH**INDEX**

Preface	20
1. Product introduction	21
1.1 Product information	21
2. Safety instructions	22
2.1 Driving notice	22
2.2 Other notice	23
3. Adjusting the compact power chair	23
3.1 Folding compact power chair	23
3.2 Armrest usage	24
3.3 Adjusting the Joystick	25
4. Battery and charger	26
4.1 Battery	26
4.2 Charger requirements	26
4.3 Charger usage	26
4.4 Battery usage and maintenance	27
5. System diagnostics	28
6. Controller	29
6.1 Controller panel	29
6.2 Controller usage	29
7. Maintenance	31
7.1 Cleaning and battery maintenance	31
7.2 Power compact chair malfunction and checking	31
7.3 Maintenance checking	32
8. Other	33
9. Warranty	33

PREFACE

Please read the user manual carefully before use.

- This manual contains operation, assembly methods, and simple fault solutions.
- This manual applies to our model Felix E.
- This manual contains compact power chair maintenance and self checking methods. Please keep it for future reference.
- Please provide this manual for reference when other people are going to use this compact power chair.
- The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design.
- Contact with your dealer if there is any ambiguity.
- Improper use of any vehicle may lead to injury. Unsafe driving could harm yourselves and others.
- The compact power chair is intended to comfortably transport persons with walking difficulties or no walking abilities.
- This compact power chair is designed to transport 1 person only.

Symbols used in this manual

Warning symbol

Follow the instructions next to this symbol closely.

Not paying careful attention to these instructions could result in physical injury or damage to the compact power chair or the environment.

1. PRODUCT INTRODUCTION

1.1 Product Information

Model	Felix E	Felix E V2
Frame	Aluminum, liquid painting	
Controller	PG	Wuyang
Motor (2)	200W LINIX	200W brushless Motor w/electromagnetic brake
Battery	Litio 20Ah	Litio de 24V 12A
Charger	HP24V3A	24V2A
Max speed	7 km/h	
Travel mileage	15 km	
Gradeability	8 degree	
Front wheel	8" PU caster	
Rear wheel	12" pneumatic tire, aluminum alloy wheel hub	
Armrest	Flip back armrest	
Footrest	Swing in and away, heel strap	
Seat	Detachable seat cushion	
Optional	Anti-tip	
User weight	136 kg. / 300 lbs.	
Product weight	36 kg. / 79.4 lbs.	
HCPCS	K0013	

Product performance

This series of power compact power chairs are powered by battery, driven by motor. Users control direction and adjust speed by joystick controller. It is suitable for application of low speed, good road condition and small slope.

Main structure

This series of compact power chairs are mainly composed of two parts:

- 1) Frame Part: frame, wheel, footrest, armrest, backrest etc.
- 2) Electrical Part: controller, motor, battery.

Application

This power compact power chair is for disabled and elderly people (less than 135kg) using as a mobility aid and is suitable for short-mid distance use. Not for use on roads as a motor vehicle.

2. SAFETY INSTRUCTIONS

2.1 Driving notice

General Driving Precautions

- Please keep your hands on the armrest to control the compact power chair.
- Please practice in parks or other safe open places until you can use the compact power chair skillfully.
- Please strictly follow pedestrian traffic rules.
- Please drive on the sidewalk and the zebra crossing. Do not drive on vehicle lane.
- Avoid sharp turns.
- Please maintain proper pneumatic tire pressure.

Avoid driving in these conditions

- Bad weather, such as rainy day, heavy fog, strong wind, snow, etc.
- If compact power chair gets wet, wipe with a towel immediately.
- Bad surface condition, such as muddy, trail, sand road, gravel, etc.
- Crowded areas.
- Hazardous areas such as steep hills, bodies of water, etc.
- Stop at any railroad crossing to confirm it is safe and that the tires will not get stuck.
- The power compact power chair is only for personal use, do not carry other people or heavy objects.

Precautions for hills and slopes

- Do not drive on slopes steeper than 9 degrees.
- Do not stop on slopes.
- Reduce speed when going down a slope.
- Do not drive across a slope.
- Do not try to drive up or down stairs.

Do not set the compact power chair to manual mode on a slope.

In the case of an emergency, please immediately ask a passerby for help. Set the compact power chair to manual mode, then push compact power chair to safety.

Precautions for the caregivers

- Caregivers should confirm that the users' feet are on the footrests, in the proper position and ensure clothes do not attach to the wheels.
- Caregivers should push the compact power chair on steep or long slopes.

2.2 Other notice

Repair and modifications

If repair is necessary, please contact your dealer. Do not modify the compact power chair by yourself. Unauthorized modification may lead to further damage or injury.

Protect the environment

In order to protect the environment, do not casually discard the battery. Please contact your local dealer for disposal and replacement of a new battery.

Storage

Do not store the compact power chair in a damp location. If the compact power chair gets wet, wipe dry immediately.

Do not add uneven weight to the compact power chair that would cause imbalance.

Do not use another vehicle to pull or push the compact power chair. If the compact power chair makes an abnormal sound, contact service immediately.


Electromagnetic interference

Keep the compact power chair away from strong magnetic fields and large inductive electrical equipment, such as radio stations, TV stations, or cell phone towers. Stay as far as away from those sources as possible to avoid electromagnetic interference.

3. ADJUSTING THE COMPACT POWER CHAIR


3.1 Folding the compact power chair

Step1: Flip lock latch upward as shown in Picture 1.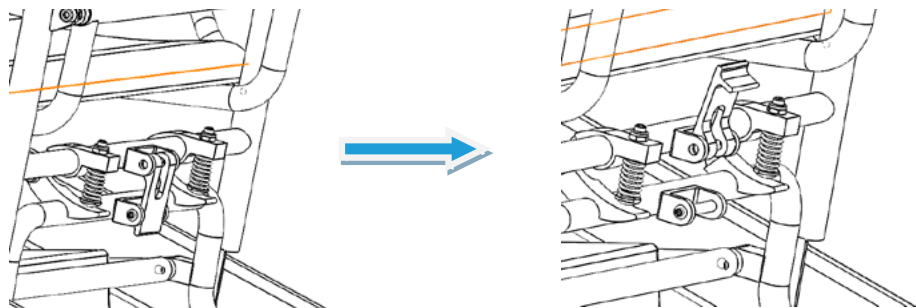

Picture 1


Step 2: Push the backrest tube down as shown in Picture 2 until completely folded.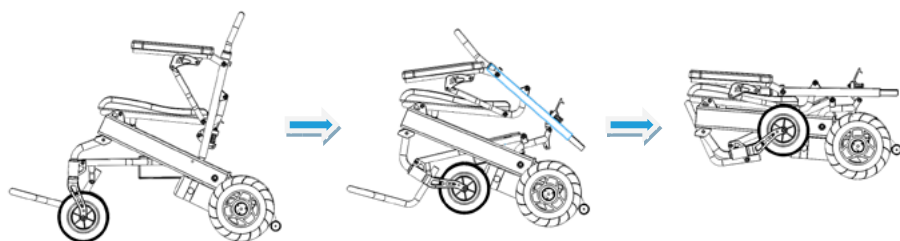


Picture 2

3.2 Armrest usage

Step1: Flip the knob switch down.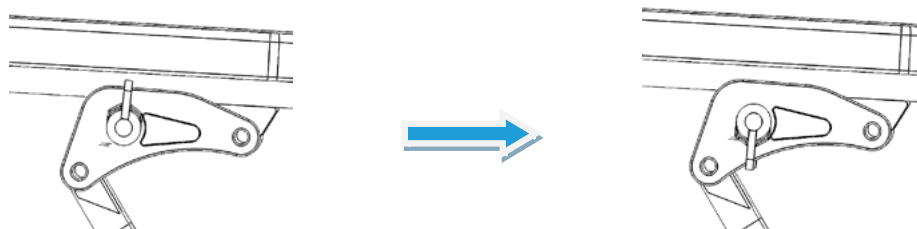

Step 2: Lift the armrest.

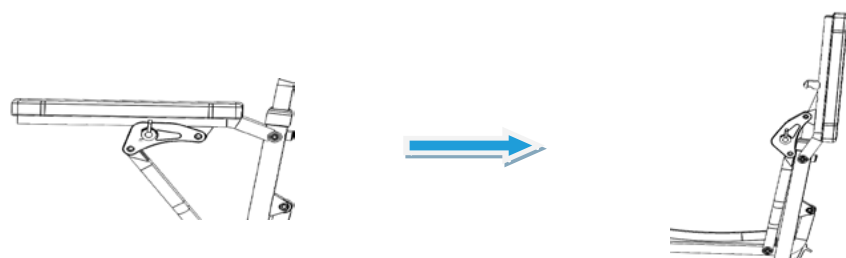3.3 Adjusting the joystick

Slide the tube attached to the controller into the tube attached below armrest. Adjust the controller to the perfect position by screwing the nut counterclockwise (picture 3). Tighten the nut when the adjusted.)

Picture 3

Switching manual and electric mode

When changing from manual to electric mode, please confirm both side clutches are inside the grooves. If either side clutch is misplaced, it may cause the compact power chair to roll over.

Manual mode:

Stop the compact power chair and turn off the power. Turn both side clutch levers until they point to point to the yellow label. (Picture 4)

Picture 4

Electric mode:

Turn both side clutch levers until they point to the green label. (Picture 5)

Picture 5

Do not put compact power chair into manual mode while driving on a slope.

4. BATTERY AND CHARGER

4.1 Battery

The compact power chair batteries come pre-installed. You do not need to remove the batteries to fold the compact power chair.

4.2 Charger requirements

Do not use the compact power chair while the charger is plugged in and charging the batteries.

Charger technical data

Input voltage: ac220v $\pm$ 10% | output voltage: 24v/dc | output current: 3a

4.3 Charger usage

To charge the battery, connect the charger to the power source and its round plug to the joystick of the chair. Follow these steps to complete the charging process:

Step 1: Ensure the charger input into the joystick is not blocked.

Step 2: Ensure the compact motorized chair is turned off.

Step 3: Connect the charger to the joystick of the chair via its round connector.

Step 4: Plug the charger's main plug into the power source and the red light will turn on. Full charging takes 8 to 10 hours, do not overcharge for more than 24 hours.

Step 5: To prevent battery life from shortening, charge it at least once a month when not using the compact motorized chair.

Do not stop charging until the process is finished. Using a battery that is not fully charged will shorten the battery life.

When the battery is fully charged, the power indicator will turn to green. Do not stop charging before fully charged.

Once charging is complete, unplug the charger from the power supply. Do not charge more than 24 hours.

Users should follow the following rules to avoid charging dangers:

Environment should be well ventilated when charging. Do not charge the compact power chair in direct sunlight or excess humidity.

Charging should be done in an environment with a temperature ranging from 10°C to 50°C. If it is done in a temperature outside of this range, the battery is likely to become damaged.

It's normal for the fan making a sound during charging.

Prevent liquid from going into the charger during charging and do not place charger on the flammable items.

4.4 The usage and maintenance of battery

Replacing the battery incorrectly may cause explosion.

Only the same or recommended type of battery is suitable for replacement.

Please make sure the battery poles are correct.

To keep battery power full, charge frequently.

It's better to fully charge the battery twice per month even if the compact power chair is not being used.

Keep open flames away from battery and charger while charging. Flames may cause battery fire or explosion.

Charging creates hydrogen. Do not smoke while charging.

Do not unplug the power supply when the socket or your hands are wet. It may cause electric shock.

To avoid injury, do not use or sit on the compact power chair while charging.

5. SYSTEM DIAGNOSTICS

When the indicator LED lights are blinking, it means there is a problem with the compact power chair. This problem can be diagnosed by the LED and audible beep pattern.

LED / Audio indication

Description of LED light	Meaning	Explanation & Solution
All LED lights are unlit with no sound	The power is off, wheelchair in standby or sleep mode. Poor battery connection. Fuse is tripping or burned out.	Try to turn on wheelchair. If no response, check battery and fuse.
All LED lights are lit	The power is turned on, self diagnosis passes, and the electric wheelchair can work well.	LED lights indicate amount of battery power remaining.
The leftmost red LED light is lit	The battery power is extremely inadequate.	Charge immediately or battery is malfunctioning and not able to charge.
Two short beeps with five indicator lights blinking twice	The left side motor is malfunctioning.	The left side motor has poor connection or the wire is disconnected.
Four short beeps with five indicator lights blinking four times	The right side motor is malfunctioning.	The right side motor has poor connection or the wire is disconnected.
Six short beeps with five indicator lights blinking six times	Controller is in over-current protection status.	Check the brakes, and check if the motor drive mechanism is stuck. Check the current using an ammeter. Insufficient current can mean the controller is malfunctioning.
Seven short beeps with five indicator lights blinking seven times	Joystick is malfunctioning.	Reset the joystick or check for a loose connection.
Eight short beeps with five indicator lights blinking eight times	Controller is malfunctioning.	Please consult your dealer for maintenance.
Nine short beeps with five indicator lights blinking nine times	Controller is malfunctioning.	Please consult your dealer for maintenance.

6. CONTROLLER

6.1 Controller panel

6.2 Controller usage

Power switch

Press this button to turn on the power. The battery power gauge lights will turn on from left to right. Press again to turn off the power. All the LED lights will turn off.

In an emergency, you can directly turn off the power by pressing power button.

Sleep mode

If the joystick has no operation for 20 minutes, the power will turn off automatically and the system will go into the sleep mode. The system will be woken up from sleep mode by pressing the power button.

Speed adjustment

Felix E

Felix E V2

According to user's habits and the circumstances, the compact power chair driving speed is adjustable. Adjust speed by pressing the decrease button or increase button. Speed is divided into five gears - 20%, 40%, 60%, 80%, 100% of the max speed. (Felix E & Felix E V2)

Horn button

Press the horn button and the horn will sound until you release the button.

Battery power gauge

After turning on the power, the battery power gauge will light up. The battery power gauge also shows the remaining battery power.

As shown, the battery power is full.

When there is only the red or yellow LED light lit, the battery needs charging. For driving long distances, the battery should be fully charged. If only the red LED light is lit, the battery power is extremely low. Charge battery as soon as possible.

Using the Joystick

The compact power chair direction is controlled by the joystick. The degree of joystick movement also controls the moving speed.

Please only turn the power on or off when the joystick is in the middle position. Otherwise, the controller will return a wrong operation signal.

Releasing the joystick to the middle should make this stop. If the issue continues, the joystick may be malfunctioning. Do not use the joystick and contact your dealer for maintenance.

7. MAINTENANCE

Maintenance of the compact power chair includes cleaning the compact power chair, checking the wheels, checking the batteries, and regularly charging the batteries. For further maintenance, please contact your dealer. Our suggestion is to check the compact power chair every six months, and annually do an overhaul.

7.1 Cleaning and battery maintenance

Cleaning the compact power chair

Please clean the compact power chair regularly.

Clean the parts which are frequently touched by the user's body (such as the cushion, armrests, joystick) by a clean and damp cloth. Do not use organic solvents to clean.

Wheels

Check the tires' air pressure and wear condition regularly. When the tire tread pattern depth reduces to 1mm, please replace with new tires.

Batteries

Make sure that the batteries are fully charged frequently. In order to prolong battery life, we strongly recommend users do not charge until the batteries run out of power totally.

7.2 Power compact chair malfunction and checking

When malfunctions happen while the compact power chair is in use, please turn off the power before checking.

Check steps:

Step 1: check if the controller's plug is loose.

Step 2: check if the connection of controller's plug and battery box is loose. Please reinsert the plug connector. If the compact power chair is still not able to restore the power, or if you have any question about the malfunction, please connect with your dealer.

The controller has a diagnosing system to monitor the controller and motor.

Any malfunction of these parts is indicated by the controller. For more details, please kindly refer to page 28.

7.3 Maintenance checking

The compact power chair should be checked according to the suggestions below.

A area	Before use, please check if following parts are in working order: 1. Backrest 2. Armrest 3. Controller position 4. Footrest 5. Battery power 6. Clutch/adjusting lever for conversion between manual mode and electric mode
B area	Check the following parts monthly for wear or loosening: 1. Screws 2. Brakes 3. Clutch/adjusting lever for conversion between manual mode and electric mode 4. The front and rear wheels and their tread pattern depth 5. Connectors’ of controller and charger
C area	For safety, semiannual overall maintenance is needed.

8. OTHER

Condition for transport and storage

During transport and storage, the compact power chair should be kept in appropriate conditions.

- During transport, avoid moisture, sunlight and heat sources.
- To avoid damaging electric parts, avoid storing the compact power chair outdoors or in damp areas.
- Storage condition:
 - Temperature -40°C~+55°C;
 - Relative humidity $\leq 80\%$;
 - Air Pressure 86kPa~106kPa.

9. WARRANTY

- 1 year warranty on the frame and parts (excluding batteries).
- 6 month warranty on batteries.

For warranty claims, please contact your local Loh Medical representative.

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