Cojín OneLoh OneLoh Cushion Almofada OneLoh

Manual de usuario en Español	
Cojín OneLoh	... 3
Owner's manual in English	
OneLoh Cushion	... 7
Manual do usuário em Português	
Almofada OneLoh	... 11

USA - Loh Medical Headquarters

✉ Info@lohmedical.com

☎ Ventas (570) 316-0872

México

✉ infomexico@lohmedical.com

☎ Ventas (52) 56 2099 9337

Aruba | Barbados | Bermuda | Curacao

Ecuador | Guatemala | Panamá

Puerto Rico | Trinidad y Tobago

✉ caribe@lohmedical.com

☎ Ventas +1 (570) 319 4647

Bahamas | Bonaire | El Salvador

Honduras | Islandia | Jamaica

Nicaragua | República Dominicana

Otros

✉ ventas@lohmedical.com

☎ Ventas +1 (570) 213 7984

Ventas +1 272 6109

Argentina | Bolivia | Chile | Paraguay

Perú | Uruguay

✉ infoconosur@lohmedical.com

☎ Ventas +1 (570) 218 4990

Ventas +54 9 3489 48-5497

Brasil

✉ infobrasil@lohmedical.com

☎ Vendas (11) 99132-1853

Colombia

✉ ventascolombia@lohmedical.com

☎ Ventas (315) 9850710

☎ Servicio técnico (315) 2355302

☎ Postventa (315) 8749566

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Costa Rica

✉ infocostarica@lohmedical.com

☎ Ventas (506) 6409 2196

☎ Servicio técnico (506) 6489 7658

☎ Atención al paciente (506) 6479 1786

☎ Ventas a Distribuidores (506) 6195 6785

318 Davis St, Ste 1, Clarks Summit, PA 18411

www.lohmedical.com

ESPAÑOL

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ENGLISH

INDEX	Specs	... 7
	Components	... 7
	General Warnings	... 7
	OneLoh Cushion	... 9
	Fitting your cushion	... 9
	Maintenance	... 9
	Assembly	... 10
	Warranty	... 10

SPECS

Materials	High Rebound Foam and Nylon
Measures	14x14", 16x16", 18x16", 18x17", 20x18"
Seat thickness	8 cm 3.1"
Lower thigh thickness	7 cm 2.7"
Medial thigh thickness	4.5 cm 1.7"
Lateral thigh support	4.5 cm 1.7"
Pelvic Contour	3.5 cm 1.3"
Weight capacity	130 kg. 286 lbs.

COMPONENTS

Please ensure that the cushion is installed by an authorized distributor of Loh Medical. Loh Medical advises consulting with a healthcare professional, such as a doctor or therapist with expertise in seating and positioning, to assess if the **OneLoh Cushion** is suitable.

GENERAL WARNINGS

WARNING: Do not attempt to install this equipment without first thoroughly reading and understanding the accompanying manual. If you find any of the Warnings, Cautions, or Instructions unclear or difficult to understand, please seek assistance from a qualified clinician or supplier to prevent the risk of injury or damage.

WARNING: Be aware that installing a cushion on a wheelchair may alter the wheelchair's center of gravity, potentially causing the wheelchair to tip backwards and result in injury. It is essential to evaluate the necessity of adding anti-tippers or amputee axle adapter brackets to the wheelchair to enhance stability.

WARNING: Before settling into prolonged sitting, any cushion should be tested for a few hours at a time while a clinician monitors your skin to prevent the development of red pressure spots.

Regularly inspect your skin for redness, as this is a clinical indicator of tissue breakdown. Should redness occur, cease using the cushion immediately and consult your doctor or therapist. Note that while the cushion is designed to mitigate pressure, it is not capable of entirely eliminating sitting pressure or preventing pressure sores.

Furthermore, the cushion is not a replacement for proper skincare practices such as a balanced diet, cleanliness, and regular pressure relief.

WARNING: Avoid exposure to sharp objects, excessive heat, and open flames.

WARNING: Do not place any obstructions between the user and the cushion, as this may impair the product's effectiveness.

WARNING: Ensure the product is used with the cushion cover facing up. Failure to use the cover correctly may diminish or nullify the cushion's benefits, increasing the risk of harm to the skin and soft tissues.

WARNING: Before settling into prolonged sitting, any cushion should be tested for a few hours at a time while a clinician monitors your skin to prevent the development of red pressure spots.

Regularly inspect your skin for redness, as this is a clinical indicator of tissue breakdown. Should redness occur, cease using the cushion immediately and consult your doctor or therapist.

Note that while the cushion is designed to mitigate pressure, it is not capable of entirely eliminating sitting pressure or preventing pressure sores. Furthermore, the cushion is not a replacement for proper skincare practices such as a balanced diet, cleanliness, and regular pressure relief.

WARNING: Avoid exposure to sharp objects, excessive heat, and open flames.

WARNING: Do not place any obstructions between the user and the cushion, as this may impair the product's effectiveness.

WARNING: Ensure the product is used with the cushion cover facing up. Failure to use the cover correctly may diminish or nullify the cushion's benefits, increasing the risk of harm to the skin and soft tissues.

OneLoh CUSHION

When selecting a cushion, make sure it matches the width of your hips while seated. The length of the cushion should extend to within 1" - 2" (2.5cm - 5cm) from the back of your knees. Position the cushion in the wheelchair with the pelvic loading area towards the back.

If fitted correctly, your hips should be at the back of the chair, with your ischials (seat bones) centered at the rear of the cushion.

Properly adjusting the footrest can improve your sitting comfort and potentially reduce peak sitting pressures.

Make sure the footrests are set so that your legs are supported firmly yet comfortably on the cushion.

FITTING YOUR CUSHION

When selecting a cushion, make sure it matches the width of your hips while seated. The length of the cushion should extend to within 1" - 2" (2.5cm - 5cm) from the back of your knees.

Position the cushion in the wheelchair with the pelvic loading area towards the back. If fitted correctly, your hips should be at the back of the chair, with your ischials (seat bones) centered at the rear of the cushion.

Properly adjusting the footrest can improve your sitting comfort and potentially reduce peak sitting pressures.

Make sure the footrests are set so that your legs are supported firmly yet comfortably on the cushion.

MAINTENANCE

Maintaining regular monthly cleanings and consistent upkeep can prolong the lifespan of your cushion. During these cleanings, it is also recommended to inspect all components of the cushion.

Examine the cover for any tears, signs of excessive wear, or other irregularities. Ensure the foam base maintains consistent foam integrity.

Cleaning the cover

1. Separate the outer cover from the foam base and zip it closed.

2. Machine wash in warm water, then either drip dry or tumble dry on a low heat setting.

Ironing is not necessary.

NOTE: Do not dry clean the cover, nor use industrial washers and dryers. Steam autoclaving and bleaching are also not recommended. After washing, place the cover back over the foam base, making sure the back of the cover aligns with the back of the foam base.

Cleaning the foam base

Remove the cover and lightly wipe the foam base with a damp cloth. Soap is not necessary and the foam should not be submerged in water. Wipe away any excess moisture with a clean cloth and let the foam base air-dry thoroughly before reassembling.

NOTE: Never submerge the foam base in water.

ASSEMBLY

After cleaning, place the foam base back into the cover, ensuring that the zipper is positioned at the back of the foam base, aligning with where the pelvis usually rests.

WARRANTY

Loh Medical offers a 12 month warranty on the frame of the Basic Cushion.

Any defective frame components will be replaced at no cost.

Please contact your local Loh Medical sales rep for warranty service. For more information on your local Loh Medical sales rep, please visit:

<https://www.lohmedical.com/en/contact>

The Basic Cuyshion has been tested and certified in accordance with CE requirements and BSEN 12183.

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