Silla de ruedas bariátrica
Bariatric Wheelchair
Cadeira de rodas bariátrica

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SPECS

Frame	Steel Folding
Seat Width	22" & 24"
Seat Depth	18"
Product Weight	24kg. / 53 lbs
Weight Capacity	150kg. / 330 lbs
Front Wheel	8"x 2" solid
Rear Wheel	24" solid
Armrest	Folding desk length PVC
Footrest	Removable & height adjustable
Fabric	Extra strong fire resistant nylon with foam
HPCPS	K0006 & K0007

USAGE

Unfolding.

Hold the handles with both hands to slightly separate the frame. Push down on both sides of the seat until it is fully unfolded.

Note: When unfolding, do not force the frame too hard to avoid damage. Do not wrap your fingers around the seat tube to avoid problems when pushing down the seat upholstery.

Folding.

First, remove the backrest stabilizer bar by turning the bolts. Then, rotate both footrests. Next, gently lift the front and rear upholstery of the seat with your hands until it folds.

Adjusting the footrest.

First loosen the locking nut with the wrench, pull down the slide tube and adjust the foot plate to a suitable position, and then lock and fix the bolts with the wrench clockwise.

Self-propelled.

While sitting on the chair, hold the hand rings with both hands.

At the same time, push the hand rings forward with equal force with both hands to do so, move forward and immediately release the grip of the hand rims.

Repeat this process until you have reached the desired distance.

Using the brake.

While stopped, the brake is operated forward until they lock into place.

This prevents the rear wheels from moving.

Pull the brake handles back to unlock the rear wheels.

Overcoming obstacles while pushing someone.

To overcome small obstacles such as low curbs or changes in terrain, the assistant pushing the chair should begin by stepping on the pedal at the end of the frame tubes while holding the push handles with both hands.

This will allow the assistant to tilt the wheelchair backward to lift the front wheels.

Once the front wheels are over the obstacle, the rear wheels can also be pushed over the obstacle.

Using the folding armrests.

To flip the armrest back, first press the armrest latch with one hand to separate it from the front seat tube bracket.

Once separated, it will freely flip back.

When you throw the armrest forward, place the latch into the tube receiver and press lightly to lock it in place.

Using the removable footrest.

When assembling the footrest, you must first align the up and down positioning holes with the pins.

Hang the footrest on the pins and then swing the footrest inward to lock it in place.

To detach the footrest, first, press the locking lever and swing the footrest outward.

Then, lift the footrest off the pins to remove it.

MAINTENANCE

Wheelchairs should be examined regularly to maintain working order.

Any loose parts should be addressed immediately.

Contact your local dealer to resolve.

Oil should be used on the moving parts of wheelchairs to maintain working order and increase the life of the chair.

Often, dry cloth can be used to wipe water and dirt on the surface of wheelchairs after use to prevent rust.

The upholstery of the seat and backrest should be kept clean to protect the skin of users.

WARNINGS

The wheelchair is used as an aid for disabled, injured and elderly people.

It is only designed for use on flat indoor or outdoor surfaces.

The user must be fit for long distance use.

This chair can load **150kg** on flat road. Do not use if the user is over **150kg**.

The brakes are used to prevent movement on slight slopes, they are not used to slow down a moving wheelchair, especially on a slope.

Do not stop on the footrests.

This may cause the chair to tip over.

While using the wheelchair, do not turn the chair or slow down suddenly.

Do not use with a flat tire. This may damage the tires and wheels.

Pay attention to the air pressure when inflating the tires to avoid burst.

The pressure should be restricted to 3.5kg/cm² on the rear wheel.

PERFORMANCE PARAMETERS

Rated load: **150 kg / 330 lbs.**

Minimum turning radius: ≤ 0.85

Static stability: (vertical) $\geq 10^\circ$, (lateral) $\geq 15^\circ$

Rear wheel diameter: 610mm

Braking effectiveness: $\geq 8^\circ$

WARRANTY

Loh Medical offers a 12 month warranty on the frame of the Coyote M.

Any defective frame components will be replaced at no cost.

Please contact your local Loh Medical sales rep for warranty service.

For more information on your local Loh Medical sales rep, please visit:

<https://www.lohmedical.com/en/contact>

The Wheelchair has been tested and certified in accordance with CE requirements and BSEN 12183.

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