FELIX

Silla motorizada plegable
Folding Power Chair
Cadeira motorizada dobrável

FELIX

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ENGLISH

INDEX

Preface	21
1. Product introduction	22
1.1 Product information	22
2. Safety instructions	23
2.1 Driving notice	23
2.2 Other notice	24
3. Adjusting the folding power chair	24
3.1 Folding the folding power chair	24
3.2 Connect wire line	25
3.3 Adjusting the Joystick	25
3.4 Assemble/Disassemble the footrest	26
3.5 Adjustment of Anti-tips	26
3.6 Switch between manual / electric	26
4. Battery & charger	27
4.1 Battery	27
4.2 Charger requirements	28
4.3 Charger usage	28
4.4 Battery usage and maintenance	29
5. System diagnostics	30
6. Controller	32
6.1 Controller panel	32
6.2 Controller usage	32
7. Maintenance	34
7.1 Cleaning and battery maintenance	34
7.2 Power compact chair malfunction and checking	34
7.3 Maintenance checking	35
8. Other	36
9. Warranty	36

PREFACE

Please read the user manual carefully before use.

- This manual contains operation, assembly methods, and simple fault solutions.
- This manual contains folding power chair maintenance and self checking methods. Please keep it for future reference.
- Please provide this manual for reference when other people are going to use this wheelchair.
- The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design.
- Contact with your dealer if there is any ambiguity.
- Improper use of any vehicle may lead to injury. Unsafe driving could harm yourselves and others.
- The folding power chair is intended to comfortably transport persons with walking difficulties or no walking abilities.
- This folding power chair is designed to transport 1 person only.

Symbols used in this manual

Warning symbol

Follow the instructions next to this symbol closely.

Not paying careful attention to these instructions could result in physical injury or damage to the folding power chair or the environment.

1. PRODUCT INTRODUCTION

1.1 Product information

Please read the user manual carefully before taking the product into use.

Model	FELIX	FELIX V2
Frame	Steel Blue color	
Controller	PG	Wuyang
Motor	LINIX (2)	
Battery	Lead acid 2x12V	
Charger	HP 24V	
Max speed	7 km/h	
Travel mileage	20 km	
Gradeability	8°	
Casters	2" x 8" solid	
Rear Wheels	12" solid	
Armrests	Flip-back	
Footrests	Swing-away	
Optional	Anti-tips	
Weight capacity	136 kg. / 300 lbs.	
Net weight without battery	36 kg. / 79.4 lbs.	
Gross weight with battery	65 kg. / 143.3 lbs.	
HCPCS	K0800 & K0815	

Product performance

This series of folding power chair are battery powered and motor driven. Users control direction and adjust speed via joystick controller.

It is suitable for use at low speed on smooth surfaces and small slopes.

Application range

The FELIX is for disabled and elderly people (less than 135kg) and used as a mobility device.

Suitable for outdoor middle distance use. Do not use on roads.

2. SAFETY INSTRUCTIONS

2.1 Driving notice

General driving precautions

- Please keep your hands on the armrest to control the folding power chair.
- Please practice in parks or other safe open places until you can use the wheelchair skillfully.
- Please strictly follow pedestrian traffic rules.
- Please drive on the sidewalk and the zebra crossing. Do not drive on vehicle lane.
- Avoid sharp turns.
- Please maintain proper pneumatic tire pressure.

Avoid driving in these conditions

- Bad weather, such as rainy day, heavy fog, strong wind, snow, etc.
- If wheelchair gets wet, wipe with a towel immediately.
- Bad surface condition, such as muddy, trail, sand road, gravel, etc.
- Crowded areas.
- Hazardous areas such as steep hills, bodies of water, etc.
- Stop at any railroad crossing to confirm it is safe and that the tires will not get stuck.
- The folding power chair is only for personal use, do not carry other people or heavy objects.

Precautions for slopes

- Do not drive on slopes steeper than 9 degrees.
- Do not stop on slopes.
- Reduce speed when going down a slope.
- Do not drive across a slope.
- Do not try to drive up or down stairs.

Do not set the wheelchair to manual mode on a slope.

In the case of an emergency, please immediately ask a passerby for help. Set the folding power chair to manual mode, then push folding power chair to safety.

Precautions for the caregivers

- Caregivers should confirm that the users' feet are on the footrests, in the proper position and ensure clothes do not attach to the wheels.
- Caregivers should push the folding power chair on steep or long slopes.

2.2 Other notice

Repair and modifications

If repair or modification is necessary, please contact your dealer. Do not modify by yourself, it may lead to accident or malfunction.

Protect environment

In order to protect environment, do not discard battery. Please contact your local dealer for replacement of new battery.

Storage

Do not store the wheelchair in a damp location. If the folding power chair gets wet, wipe dry immediately.

Do not add uneven weight to the folding power chair that would cause imbalance.

Do not use another vehicle to pull or push the folding power chair.

If the folding power chair makes an abnormal sound, contact service immediately.

Electromagnetic interference

Keep the folding power chair away from strong magnetic fields and large inductive electrical equipment, such as radio stations, TV stations, or cell phone towers. Stay as far as away from those sources as possible to avoid electromagnetic interference.

3. ADJUSTING THE TRANSPORTABLE POWER CHAIR

3.1 Folding the folding power chair

Step 1: Take off the battery and unplug the red plug (figure 1).

Figure 1

Step 2: Lift up the cushion strongly by hands, fold wheelchair (figure 2).

Figure 2

3.2 Connect wire line

Connect the battery box's plug to controller's plug (figure 3).

Figure 3

3.3 Adjustment of Joystick

Adjust the controller to perfect position by screwing the nut counterclockwise (figure 4). Screw tight the nut when adjustment is done.

Figure 4

3.4 Assemble/Disassemble the footrest

Install the footrest into the frame tube, and move from outside to medial to the fixed position; For disassemble, push the plastic part to outside then lift up footrest (figure 5).

Figure 5

3.5 Adjustment of Anti-tips

Adjust anti-tip angle by unscrewing the nut (figure 6).

Figure 6

3.6 Switch between manual / electric

When changing from manual mode to electric mode, please confirm both side clutches are inside the grooves.

Any side clutch is not fully into groove, may cause danger of rollover.

Manual mode:

Turn off power and stop the folding power chair, spin the adjusting levers, make it point to the yellow label (figure 7).

Electric mode:

Spin the adjusting levers, and make it point to the green label (figure 8).

Figure 7

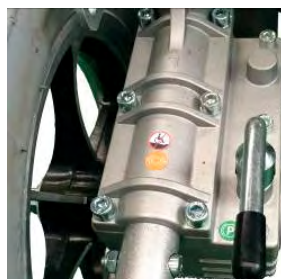
Figure 8

Do not switch the folding power chair seat to manual mode when the user is driving on a road.

4. BATTERY & CHARGER

4.1 How to assemble the battery box

Step 1: Put the battery in the battery box and fix the bolts (figure 9). Fix bolts labeled with "A" from the top down (figure 10) and bolts labeled with "B" from the bottom up (figure 11).

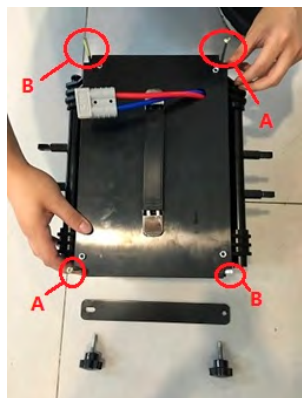
Figure 9

Figure 10

Figure 11

Step 2: Fix the battery box into the frame. Attach battery box bracket via thumbscrews (figure 12) and connect battery box to motor (figure 13).

Figure 12

Figure 13

4.2 Charger requirements

Do not use the folding power chair while the charger is plugged in and charging the batteries.

Charger technical data

Input voltage: ac220v $\pm 10\%$ | output voltage: 24v/dc | output current: 4A

4.3 Charger usage

To charge the battery, connect the charger to the power source and its round plug to the joystick of the chair. Follow these steps to complete the charging process:

Step 1: Ensure the charger input into the joystick is not blocked.

Step 2: Ensure the folding power chair is turned off.

Step 3: Connect the charger to the joystick of the chair via its round connector.

Step 4: Plug the charger's main plug into the power source and the red light will turn on. Full charging takes 8 to 10 hours, do not overcharge for more than 24 hours.

Step 5: To prevent battery life from shortening, charge it at least once a month when not using the folding power chair.

Do not stop charging until the process is finished. Using a battery that is not fully charged will shorten the battery life.

When the battery is fully charged, the power indicator will turn to green. Do not stop charging before fully charged.

Once charging is complete, unplug the charger from the power supply. Do not charge more than 24 hours.

Users should follow the following rules to avoid charging dangers:

Environment should be well ventilated when charging. Do not charge the folding power chair in direct sunlight or excess humidity.

Charging should be done in an environment with a temperature ranging from 10°C to 50°C. If it is done in a temperature outside of this range, the battery is likely to become damaged.

It's normal for the fan making a sound during charging.

Prevent liquid from going into the charger during charging and do not place charger on the flammable items.

4.4 The usage and maintenance of battery

Replacing the battery incorrectly may cause explosion.

Only the same or recommended type of battery is suitable for replacement.

Please make sure the battery poles are correct.

To keep battery power full, charge frequently.

It's better to fully charge the battery twice per month even if the folding power chair is not being used.

Keep open flames away from battery and charger while charging. Flames may cause battery fire or explosion.

Charging creates hydrogen. Do not smoke while charging.

Do not unplug the power supply when the socket or your hands are wet. It may cause electric shock.

To avoid injury, do not use or sit on the folding power chair while charging.

5. SYSTEM DIAGNOSTICS

When the indicator LED lights are blinking, it means the folding power chair is experiencing errors. The errors may occur in motors, brakes, battery, wire connections, etc.

The condition of the error can be detected by audio signal. No other service tools are needed.

See page 31 for audio signal indicator table.

LED Description	Meaning	Explanation / Solution
All LED lights are unlit with no sound	The power is off, folding power chair in standby or sleep mode. Power is poorly contacted. Fuse is tripping or burned out.	
All LED lights are lit	The power is turned on, and self diagnose is passed, the folding power chair is working	Fewer LED lights lit, less battery power remaining
The leftmost red LED light is lit	The battery power is extremely inadequate	Charge immediately or battery is malfunctioning and is not able to charge
2 short beeps and 5 indicator lights blink 2x	The left side motor is malfunctioning	The left side motor is poorly connected or the wire is disconnected
4 short beeps and 5 indicator lights blink 4x	The right side motor is malfunctioning	The right side motor is poorly connected or the wire is disconnected.
6 short beeps and 5 indicator lights blink 6x	Controller is in over-current protection status	Check the brakes, and check for stuck motor drive mechanism. Check the current by ammeter. If insufficient, the controller is malfunctioning
7 short beeps and 5 indicator lights blink 7x	Joystick is malfunctioning	Reset joystick or the connection is loose
8 short beeps and 5 indicator lights blink 8x	Controller is malfunctioning	Please consult your dealer for maintenance
9 short beeps and 5 indicator lights blink 9x	Controller is malfunctioning	Please consult service center for maintenance

6. CONTROLLER

6.1 Controller panel

6.2 Controller usage

Power switch

Press this button to turn on the power. The battery power gauge lights will turn on from left to right. Press again to turn off the power. All the LED lights will turn off.

In an emergency, you can directly turn off the power by pressing power button.

Sleep mode

If the joystick has no operation for 20 minutes, the power will turn off automatically and the system will go into the sleep mode. The system will be woken up from sleep mode by pressing the power button.

Speed adjustment


FELIX

FELIX V2

According to user's habits and the circumstances, the folding power chair driving speed is adjustable. Adjust speed by pressing the decrease button or increase button. Speed is divided into five gears - 20%, 40%, 60%, 80%, 100% of the max speed. (FELIX & FELIX V2)

Horn button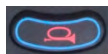

Press the horn button and the horn will sound until you release the button.

Battery power gauge

After turning on the power, the battery power gauge will light up. The battery power gauge also shows the remaining battery power.

As shown, the battery power is full.

When there is only the red or yellow LED light lit, the battery needs charging. For driving long distances, the battery should be fully charged. If only the red LED light is lit, the battery power is extremely low. Charge battery as soon as possible.

Using the Joystick

The folding power chair direction is controlled by the joystick. The degree of joystick movement also controls the moving speed.

Please only turn the power on or off when the joystick is in the middle position. Otherwise, the controller will return a wrong operation signal.

Releasing the joystick to the middle should make this stop. If the issue continues, the joystick may be malfunctioning. Do not use the joystick and contact your dealer for maintenance.

7. MAINTENANCE

Maintenance of the folding power chair includes cleaning the folding power chair, checking the wheels, checking the batteries, and regularly charging the batteries. For further maintenance, please contact your dealer. Our suggestion is to check the folding power chair every six months, and annually do an overhaul.

7.1 Cleaning and battery maintenance

Cleaning the folding power chair

Please clean the folding power chair regularly.

Clean the parts which are frequently touched by the user's body (such as the cushion, armrests, joystick) by a clean and damp cloth. Do not use organic solvents to clean.

Wheels

Check the tires' air pressure and wear condition regularly. When the tire tread pattern depth reduces to 1mm, please replace with new tires.

Batteries

Make sure that the batteries are fully charged frequently. In order to prolong battery life, we strongly recommend users do not charge until the batteries run out of power totally.

7.2 Power compact chair malfunction and checking

When malfunctions happen while the folding power chair is in use, please turn off the power before checking.

Check steps:

Step 1: check if the controller's plug is loose.

Step 2: check if the connection of controller's plug and battery box is loose. Please reinsert the plug connector. If the folding power chair is still not able to restore the power, or if you have any question about the malfunction, please connect with your dealer.

The controller has a diagnosing system to monitor the controller and motor.

Any malfunction of these parts is indicated by the controller. For more details, please kindly refer to page 31.

7.3 Maintenance checking

The folding power chair should be checked according to the suggestions below.

A area	Before use, please check if following parts are in working order: 1. Backrest 2. Armrest 3. Controller position 4. Footrest 5. Battery power 6. Clutch/adjusting lever for conversion between manual mode and electric mode
B area	Check the following parts monthly for wear or loosening: 1. Screws 2. Brakes 3. Clutch/adjusting lever for conversion between manual mode and electric mode 4. The front and rear wheels and their tread pattern depth 5. Connectors' of controller and charger
C area	For safety, semiannual overall maintenance is needed.

8. OTHER

Condition for transport and storage

During transport and storage, the folding power chair should be kept in appropriate conditions.

- During transport, avoid moisture, sunlight and heat sources.
- To avoid damaging electric parts, avoid storing the folding power chair outdoors or in damp areas.
- Storage condition:
 - Temperature $-40^{\circ}\text{C}\sim+55^{\circ}\text{C}$;
 - Relative humidity $\leq 80\%$;
 - Air Pressure $86\text{kPa}\sim 106\text{kPa}$.

9. WARRANTY

- 1 year warranty on the frame and parts (excluding batteries).
- 6 month warranty on batteries.

For warranty claims, please contact your local Loh Medical representative.

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