JAY[®]

J3 BACK

JAY[®] J3 Back

IMPORTANT CONSUMER INFORMATION

NOTICE: This manual contains important instructions that must be passed on to the user of this product.
Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: Visit www.SunriseMedical.com to register this product.

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Manuel d'instructions

ENGLISH

Introduction	3
Maintenance and Cleaning	4
Fitting the Back to the Chair.....	5
Installation	6
Adjustment.....	9
Accessories.....	10
Transit Safety	12
Removal and Replacement.....	13
Warranty	13

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Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a J3 Back is appropriate. Backs should only be installed by an authorized Sunrise Medical supplier.

THE JAY J3 BACK

The Jay J3 Back is designed to provide the important benefits of a proper back support for a wide variety of users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

Maximum user weight:

300 lbs. (136 kg)

500 lbs. (227 kg) for sizes 22", 24" and 26" widths

⚠ WARNING

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

⚠ WARNING

Do not use J3 Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.






⚠ WARNING

Do not use the J3 back handle to lift wheelchair. It is intended only for back removal.

J3 BACK

Mid Thoracic (example)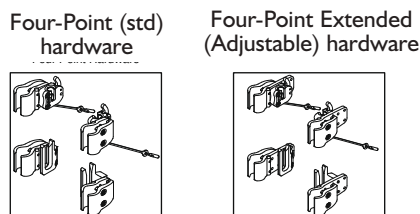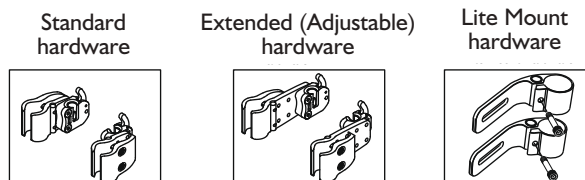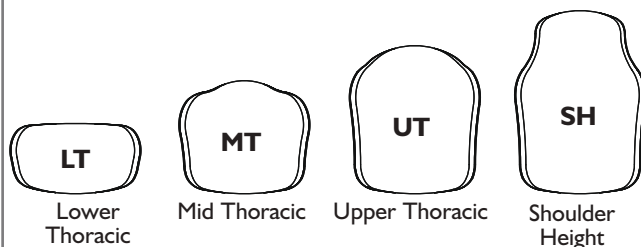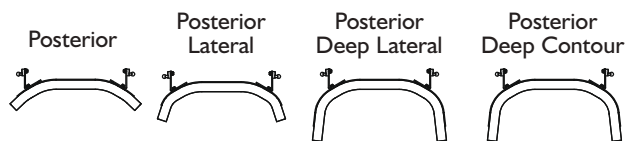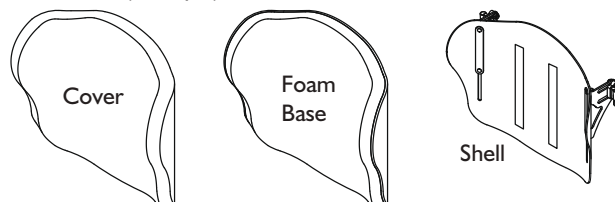

MAINTENANCE

Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be retightened according to the installation instructions. All fasteners should be tightened to torque specifications as stated in the hardware installation section. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

EASY CLEANING

NOTE– When disassembling the back for cleaning, note the position of the Velcro® Spine Align Components for correct reassembly.

To clean back cover

Remove cover by detaching Velcro. Detach cover from back shell. Locate zipper on bottom of the cover and unzip. Remove foam and turn cover inside out. Zip cover closed to avoid snagging. Machine wash in warm water (50°C). Drip or tumble dry on low heat.

NOTE: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

Reinsert the foam into the cover. Please note, the base has two layers of foam. When reinserting the foam, ensure that the thinner “light grey” foam is facing the shell. Once the foam is secured inside the cover, zip cover closed.

To clean back shell

Shell may be wiped clean with soap and water. No other maintenance should be required.

To clean positioning components

Positioning components can be wiped clean with soap and water. Do not immerse positioning components underwater.

If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

Laundry Care Symbols	
	Machine wash in warm water (50° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

DETERMINING COMPATIBILITY

Wheelchair Types

The J3 Back is designed to be compatible with most wheelchairs with the following exceptions:

Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

⚠ WARNING

Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

Wheelchair Dimensions

The J3 Back is designed to replace the wheelchair's sling upholstery. Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in Figure 1. Then using that measurement, check Table 1 to determine if the back will fit your chair.

Next check that the back canes are a compatible diameter. The J3 back hardware can be mounted on wheelchair back canes ranging from 3/4" to 1 1/8". Measure your back cane diameter (Fig. 2) and refer to Table 2 to ensure your back canes are compatible.

If you cannot determine that the back canes are compatible please contact your authorized Sunrise Medical provider or Sunrise Medical customer service.

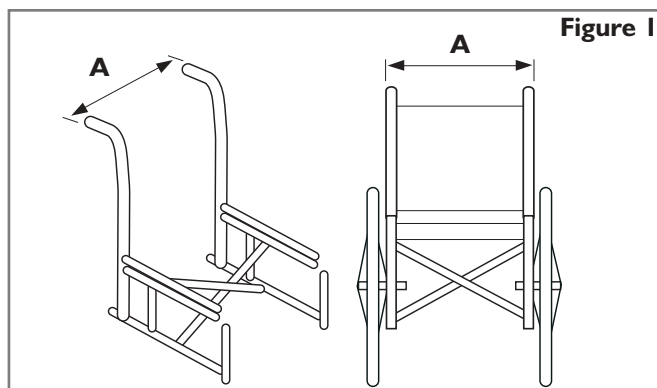
Figure 1

Table 1

J3 Back Width* (all contours)	Width of back canes from outside to outside of the tubes (Fig. 1-A)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
12"	11.7	29.7	14.3	36.3
14"	13.7	34.9	16.3	41.3
16"	15.7	40.0	18.3	46.4
18"	17.7	45.1	20.3	51.5
20"	19.7	50.1	22.3	56.5
22"	21.7	55.1	24.3	61.7
24"	23.7	60.2	26.3	66.8
26"	25.7	65.3	28.3	71.9

* Back Width is identified in your back's part number.
Example: J3PAMTS **14** SH is a 14" back.

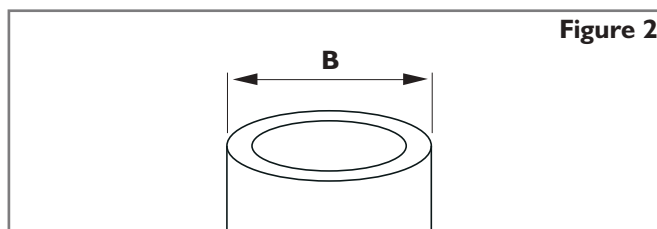
Figure 2

Table 2

J3 Back Width (all contours)	Cane diameter (Fig. 2-B)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
12" - 26"	3/4	1.9	1 1/8	2.85

INSTALLING THE MOUNTING HARDWARE

Please read the following instructions before beginning installation.

To install the J3 Back, it is best to begin without a user in the wheelchair. Once the back has been installed and minor adjustments are needed the user can be seated back in the wheelchair.

Tools required (included with J3 Back)

- 4mm hex key
- 10mm box end wrench

Hardware Installation Instructions (Figs. 3, 4, 5)

1. Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.
2. Visually determine the desired location of the receivers on the wheelchair's back canes. The mounting hardware should be at approximately equal heights on each back post (Fig. 3) and parallel to the seat frame (Fig. 4). Higher locations on the back canes are generally better.
3. Attach the hardware receivers. (Fig. 5)
 - a. Loosen the clamp screws (A) with the enclosed 4mm hex key. NOTE— For large tube diameters, clamp screws may have to be removed completely.
 - b. Attach and align receiver to the wheelchair back canes (B).
 - c. Hand tighten clamp screws (A). NOTE— Receivers may require additional lateral adjustment to ensure proper alignment.

Attaching and Aligning the J3 Back Shell (Fig. 6)

- a. Loosen bracket nuts (C) and mounting pin bolts (F) using 10mm box-end wrench until hardware can move easily in all directions.
- b. Attach the J3 Back to the hardware receivers by inserting left and right mounting pins (D) in the left and right receivers (E).
- c. Locate back to the approximate location, relative to both chair and user.
- d. Tighten bolts on the inside of the mounting pin (F). The recommended torque specification is 100 - 110 in-lbs. (11.3 - 12.4 Nm). NOTE— Back depth and angle may require additional adjustment to ensure proper fit to the user.
- e. Tighten bracket nuts (C). The recommended torque specification is 75 - 85 in-lbs. (8.5 - 9.3 Nm). NOTE— Back height may require additional adjustment to ensure proper fit to the user.
- f. Test back release by pressing the release levers forward and removing the back off the chair. (See Removal and Replacement section for further details.) Proper alignment of the mounting hardware and back is achieved when smooth attachment and release can be easily done.
- g. If proper alignment has not been achieved, adjust the components until properly aligned.
- h. Once properly aligned, tighten the hardware receiver clamp screws. The recommended torque specification is 75 - 85 in-lbs (8.5 - 9.3 Nm).

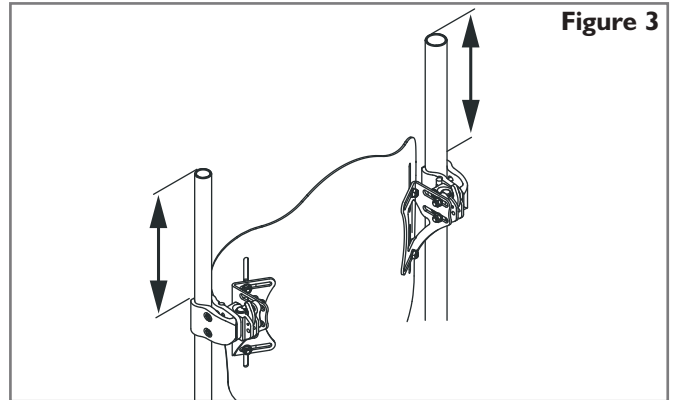
Figure 3

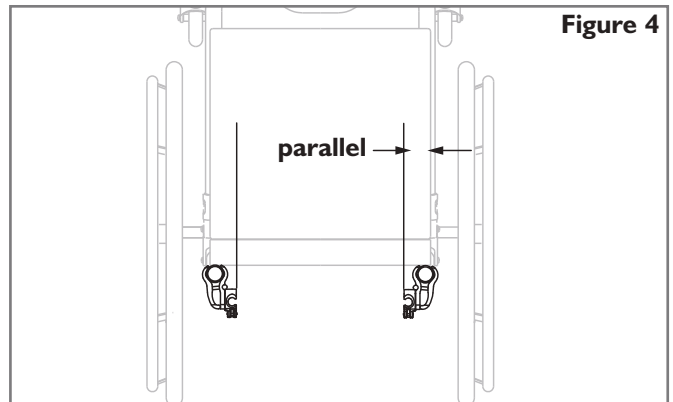
Figure 4

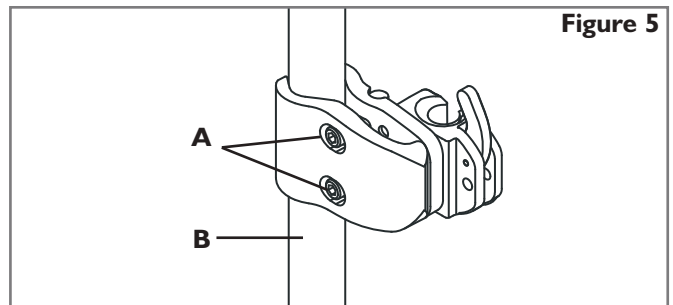
Figure 5

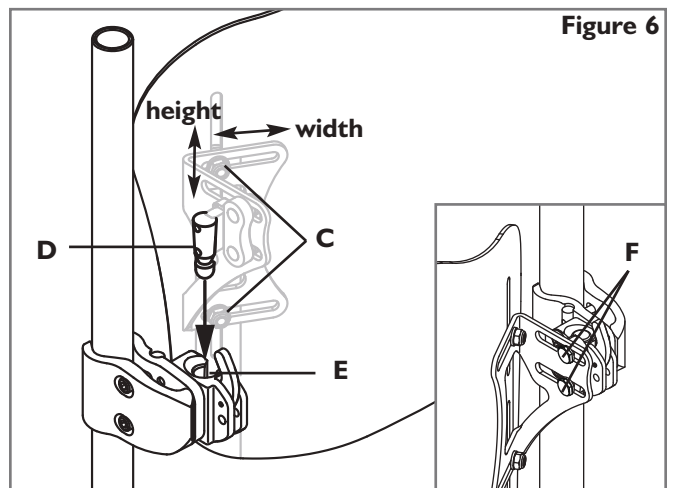
Figure 6

ATTACHING FOUR POINT HARDWARE (OPTIONAL) (FIG. 7, 8, 9)

Tools required (included with J3 Back with 4-Point Hardware)

- 4mm hex key

- 10mm box end wrench

- 5/8" box end wrench

Installing the 4-Point Hardware

1. Loosen the clamp screws (A) with the supplied 4mm hex key and slip the clamps around the Back Canes (B).
2. The mounting hardware should be at equal heights on each back post (Fig. 8) and parallel to the seat frame (Fig. 7).
3. Hand tighten clamp screws (A). Loosen bracket nuts (C) and Mounting Pin bolts (F) using the supplied 10mm box-end wrench until hardware can move easily in all directions

NOTE— For large tube diameters, clamp screws may have to be removed completely.

4. Attach the J3 Back by inserting left and right Mounting Pins (D) in the left and right Latch Receivers (E).
5. Install back to the approximate location, relative to both chair and user.
6. Tighten bolts on the inside of the Mounting Pin (F). Torque to 100 - 110 in-lbs.(11.3 - 12.4 Nm).

NOTE— Receivers may require additional lateral adjustment to ensure proper alignment.

7. Tighten bracket nuts (C). Torque to 75 - 85 in-lbs. (8.5 - 9.3 Nm).

NOTE— Back height may require additional adjustment to ensure proper fit to the user.

8. Test back release by pressing the release levers (L) forward and removing the back off the chair. Proper alignment of the mounting hardware and back is achieved when smooth attachment and release can be easily done.
9. If proper alignment has not been achieved, adjust the components until properly aligned.
10. Once properly aligned, tighten the hardware receiver clamp screws (A). Torque to 75 - 85 in-lbs (8.5 - 9.3 Nm)
11. Loosen bolts (G) to install Lower Receivers on Back Canes (H).
12. Use a 5/8" end wrench to loose the nut (K) that secures the Lower Mounting Pin (I) so that it slides fore and aft.
13. Slide lower receiver up (J) until it surrounds the Lower Mounting Pin (I).

NOTE— Leave 1/8" to 1/4" of space between the bottom of the Lower Mounting Pin (I) and the bottom of the Lower Receiver Channel (J).

NOTE— Make sure the Upper Mounting Bracket and Lower Receiver Bracket are in alignment on the Back Canes. Misalignment of the brackets may result in impaired function of quick-release.

14. Once properly aligned, tighten the hardware receiver clamp screws (G). The recommended torque specification is 75 - 85 in-lbs (8.5 - 9.3 Nm). Then tighten the Lower Mounting Pin nut (K). Torque to 100 - 110 in-lbs.(11.3 - 12.4 Nm).

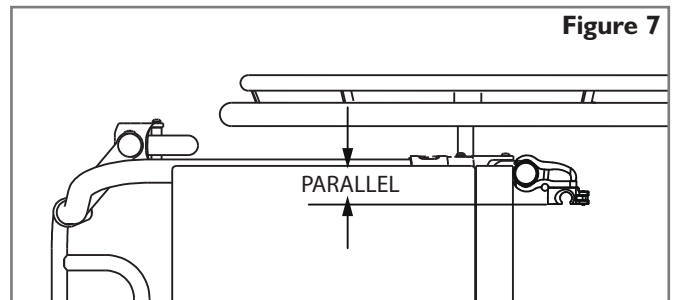
Figure 7

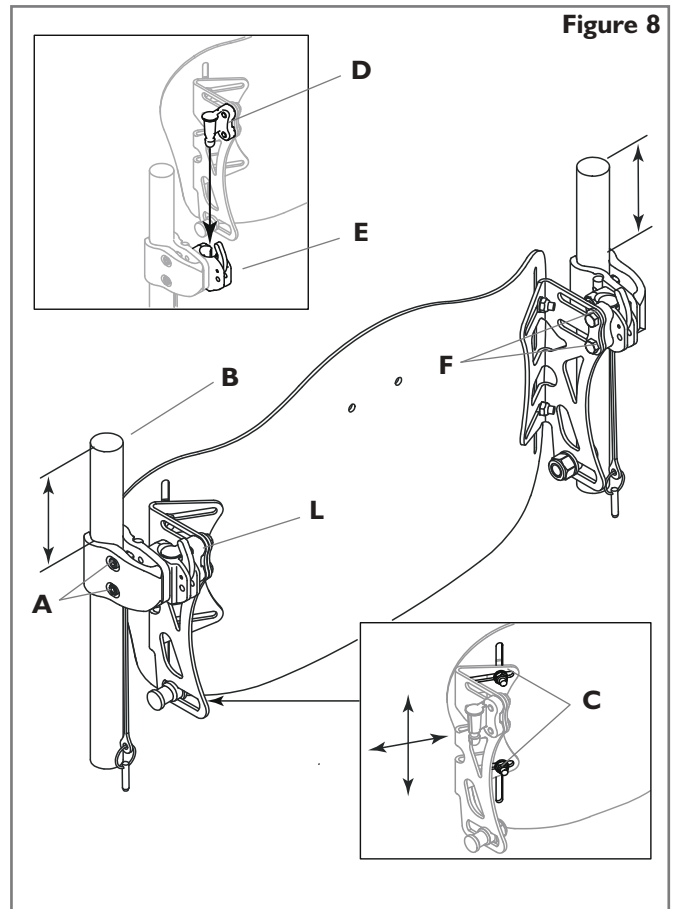
Figure 8


15. Test back release by pressing the release levers (L) forward and removing the back off the chair. (See Removal and Replacement section for further details.) Proper alignment of the mounting hardware and back is achieved when smooth attachment and release can be easily done.
16. If proper alignment has not been achieved, adjust the components until properly aligned.
17. To make micro adjustments, loosen bolts C and F. This will allow for minor changes in height, depth and angle. Retighten all bolts to required specs and test back function and quick-release.
18. Or, to make macro adjustments, loosen, but do not remove, bolts G, C and F (Figs. 8, 9). Position backrest to desired height, depth and angle.
19. Retighten all bolts to required specs and test back function and quick-release.

Adjusting the Four-Point Hardware

To make micro adjustments, loosen bolts C and F. This will allow for minor changes in height, depth and angle. Retighten all bolts to required specs and test back function and quick-release.

Or, to make macro adjustments, loosen, but do not remove, bolts G, C and F (Figs. 8, 9). Position backrest to desired height, depth and angle. Retighten all bolts to required specs and test back function and quick-release.

Figure 9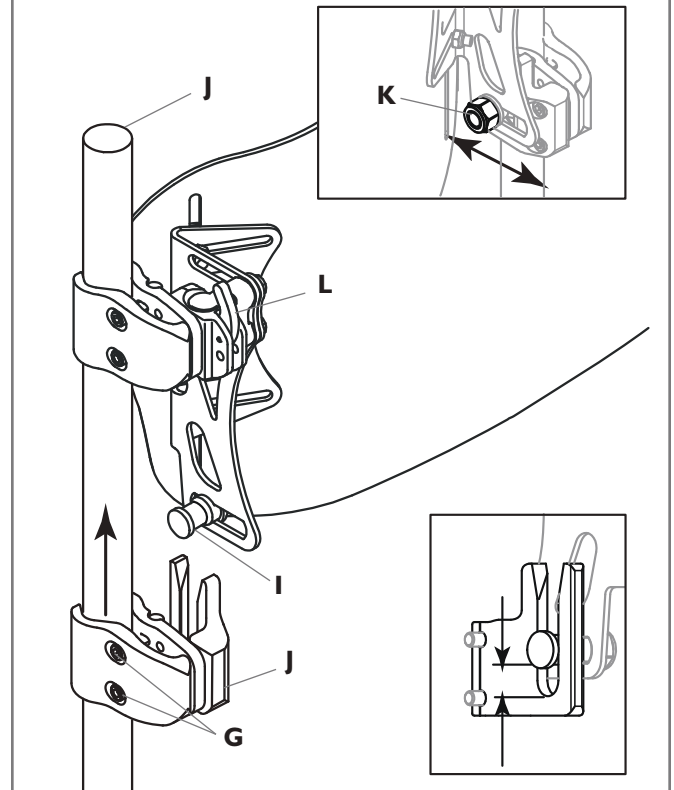

INSTALLING THE LITE MOUNT HARDWARE (FIG. 10, 11)

1. Loosen bolt (A) in each bracket.
2. Slide all four brackets onto backrest frame tubes (B).

NOTE— All Lite Mount Hardware ships with optional spacers (C) to be used as needed on back posts smaller than one inch in diameter.

3. Install backrest onto Lite Mount Hardware using bolt (D).
4. Adjust height and position of backrest and tighten.

NOTE— On backrest frame tubes with permanently attached or formed push handles, the back frame tubes may need to be removed and the mounting hardware slid from the bottom.

Figure 10


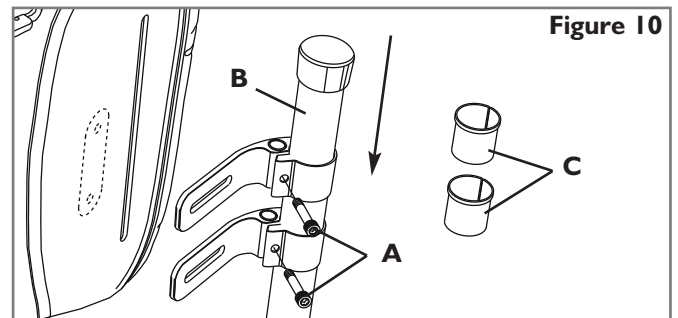
Figure 11

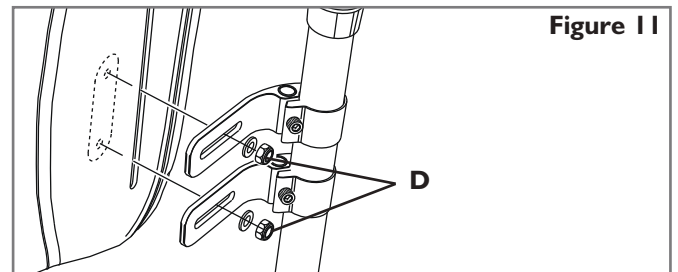ADJUSTING J3 HEIGHT, DEPTH AND ANGLE

The first phase of the J3 Back hardware installation is complete. It is now the appropriate time to have the user seated back in the chair for personalizing the J3 Back. Do not have the user apply full pressure on the back as it is not fully tightened to the full torque requirements.

1. Height Adjustment (Fig. 12)

- With the user in the chair, identify the desired height of the back. Once identified, have the user remove pressure from the J3 Back by leaning or sliding partially forward if possible to allow the adjustment to take place.
- Loosen the bracket nuts (A) and adjust the height of the back shell to required position.
- Securely tighten the bracket nuts (A). The recommended torque specification is 75-85 in-lbs. (8.5 - 9.3 Nm).

NOTE— After height adjustment is complete, check that the width was not inadvertently changed.

2. Depth/Angle Adjustment (Figs. 13, 14, 15, 16)

Please note there are two types of hardware that can be ordered—standard hardware (Fig. 13) or the longer, adjustable hardware (Fig. 14). Both sets of hardware are adjusted the same, but the standard hardware allows for 2 inches of depth adjustment (Fig. 13) while the extended hardware (adjustable hardware) allows for a total of 3.4 inches of depth adjustability (Fig. 14).

- To adjust J3 Back depth and angle loosen left and right bolts (Fig. 16-B) on bracket slots.
- Adjust depth and/or angle and securely tighten. The recommended torque specification is 100-110 in-lbs. (11.3 - 12.4 Nm). Ensure that none of the depth adjustment nuts (Fig 16-B) will slip or rotate.
- If additional depth is needed, remove back from receivers. Remove screws (Figs. 13-D, 14-D) from receiver using the enclosed 4mm hex key, and reposition to desired depth. Tighten screws to 25-35 in-lbs. (2.8 - 4 Nm).

WARNING

Ensure that all hardware has been properly tightened once all adjustments have been completed and the J3 Back is ready for use.



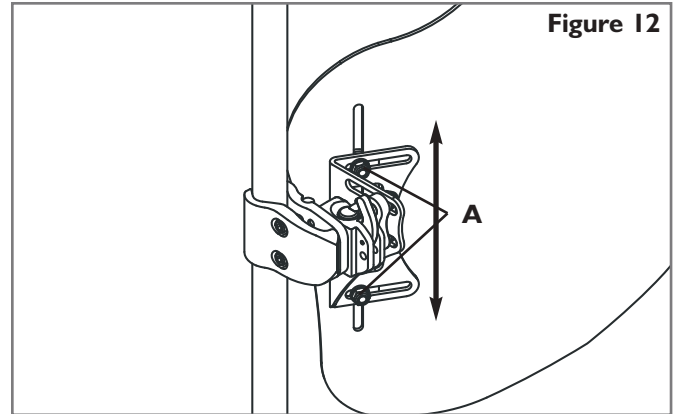
Figure 12

Standard hardware— 2 depth settings

Figure 13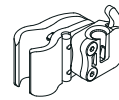


Depth 1

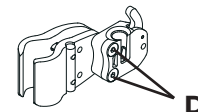
Depth 2

Extended (Adjustable) hardware


Figure 14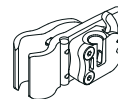


Depth 1

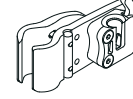Depth 2

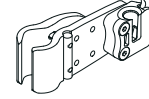Depth 3

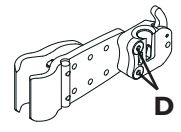Depth 4

Four-Point Hardware Lower Receivers

Figure 15

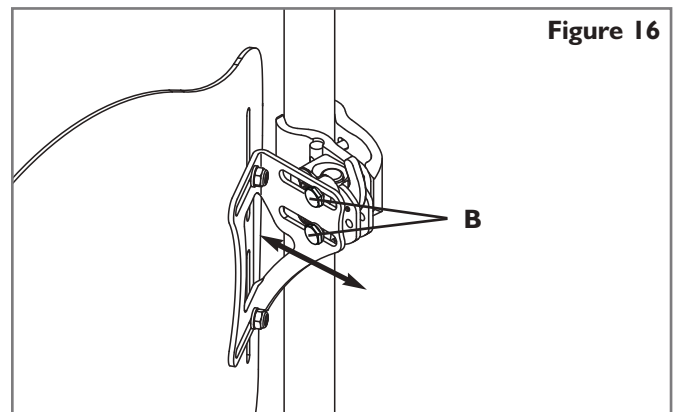
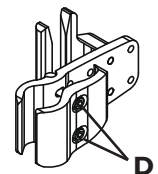
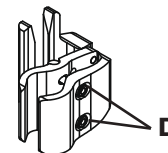
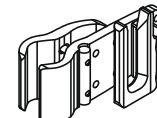
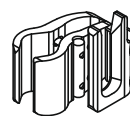
Figure 16

VANITY FLAP INSTALLATION

1. Lift upper flap of J3 Back Cover (Fig. 17)
2. Match hook to loop and attach Vanity Flap to the back of the J3 Back Cover (Fig. 17). Reposition upper flap of cover to sandwich the Vanity Flap between the flaps the upper and lower flaps (Fig. 18).
3. Remove Seat Cushion from wheelchair and sandwich “tail” of Vanity Flap between wheelchair support surface and Seat Cushion (Fig 20).
4. Adjust Vanity Flap as necessary by moving the tail in or out from under the Seat Cushion. If necessary the upper part of the vanity flap can be moved up or down on the Seat Back also.
5. Insert Hook & Loop Wrap ties through the appropriate grommets on the Vanity Flap, then wrap around the wheelchair frame and insert the leading end through the slot end of the Wrap (Fig. 21). Snug up and close outside gaps in Vanity Flap (Fig. 22).

**Figure 17****Figure 18****Figure 19****Figure 20****Figure 21****Figure 22****Figure 23****Finished J3 Vanity Flap Installation**

SPINE ALIGN POSITIONING COMPONENTS

Spine Align Positioning Components are designed to provide intimately shaped support within an “off-the-shelf” back. (Depending on your market, Spine Align components may be included with your back.) To install Spine Align please read the following directions.

I. Spine Align Positioning Component Installation

- Have the user remove pressure from the J3 Back by leaning or sliding partially forward if possible.
- Release Velcro strip on the top cover flap (Fig. 24) and lift the cover forward to reveal the inside of the J3 Back.
- Place the positioning component(s) on the Velcro inside of the shell (Fig. 25) and reattach the top cover flap. Pull down making certain that the top seam is aligned with the top of the shell. If seam does not line up, the bottom of the cover may need to be loosened (Fig. 26).
- Have the user test the current Spine Align setup for comfort and support and repeat steps a-c until adjustment is complete.
- Multiple configurations and combinations of Spine Align components can be used. If additional Spine Align shapes or sizes are required, please contact your dealer or Sunrise Medical at 800-333-4000.

NOTE– If you require further information on the J3 Spine Align system, please review our animation at www.sunrisemedical.com.

Installation of the J3 back is complete.

Positioning Components	
Large Lumbar	
Small Lumbar	
Wedge	
Curved Shim	
Flat Shim	
Posterior/Lateral Pelvic	
Lateral Shim	
Large Bean Bag	
Small Bean Bag	

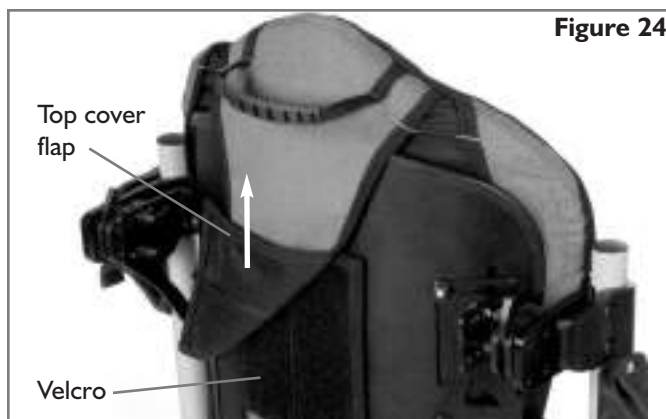
Figure 24

Figure 25

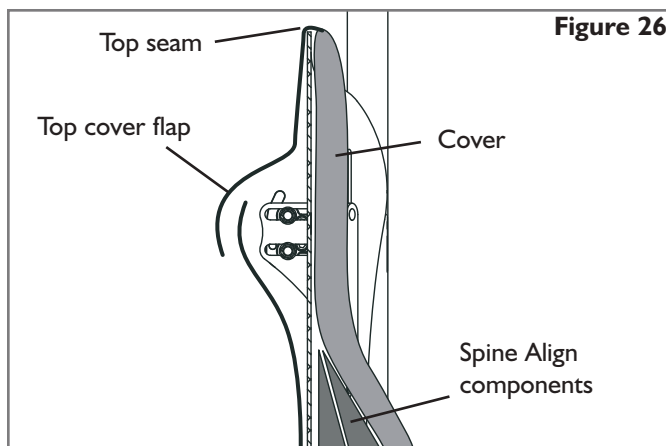
Figure 26

J3 ACCESSORIES


If you have ordered J3 accessories such as laterals, headrests or harnesses please consult the installation instructions related to those items.

BACK TRANSIT SAFETY


The J3 back has been dynamically tested for use in a motor vehicle. Please follow all installation, use and maintenance instructions within this manual as well as the transit instructions listed below.

WARNINGS

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
2. The distance between the top of the user's shoulder and the top of the back should not exceed 6.5" (16.5 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. The Wheelchair Tiedown and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
6. The wheelchair must be forward facing during transport.
7. In order to reduce the potential of injury to vehicle occupants wheelchair-mounted accessories such as trays and respiratory equipment should be removed and secured separately.
8. This product is intended for use by adults ranging in weight from 79 to 300lbs or up to 500 lbs. (227 kg) for widths over 22".
9. Postural support devices such as pelvic positioning belts, anterior trunk supports or postural supports such as lateral trunk supports should not be relied on for occupant restraint in a moving vehicle unless they are labeled as conforming to ISO Standard 16840-4.
10. Alterations or substitutions should not be made to the structure, parts or components, without consulting the manufacturer.
11. Backshell mounting bracket must be attached in the "leg up" position for transit use. (See below.) Refer to the appropriate section for correct back hardware mounting instructions.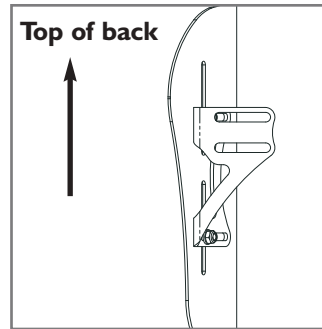

Correct bracket
orientation for transit

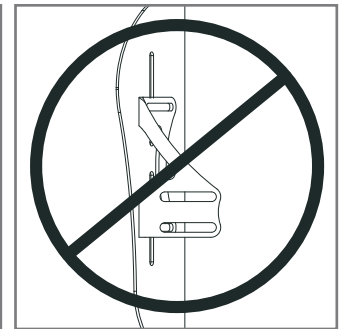Incorrect bracket
orientation for transit

⚠ WARNING

**FAILURE TO HEED THESE WARNINGS
COULD CAUSE SEVERE INJURY TO THE
WHEELCHAIR RIDER OR OTHERS.**

REMOVAL AND REPLACEMENT

Removal

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. To remove, push each release lever forward to unlock (Fig. 27). Once both levers are in the forward position, simply lift the back straight up (Fig. 28) using the handle located on the top of the back.

Proper reattachment after removal




To reattach the back to the hardware receivers, simply line up the pins to the receivers and push into place (Fig. 28). If properly inserted the pins will “click” to a locked position.

WARNING

If you are uncertain if you have heard the locking “click”, pull up on the back using the handle to ensure that it has been securely locked.

Locking Pins (Fig. 30)

The J3 Back comes with two locking pins which, when secured, prevent the back from being released from the chair. To lock, insert the pins (A) into the mounting receivers (B). Simply remove the pins if the quick-release capability is desired.

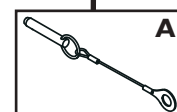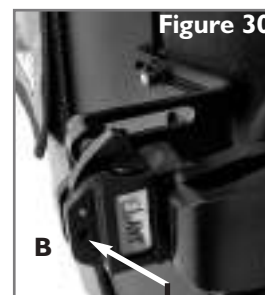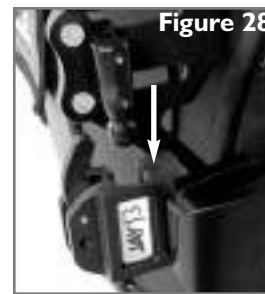WARRANTY

24 MONTH LIMITED WARRANTY

Each Jay J3 Back is carefully inspected and tested to provide peak performance. Every Jay J3 Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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©2013 Sunrise Medical (US) LLC
Sunrise Medical Inc. • 2842 Business Park Ave. • Fresno, CA 93727 • USA