maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

⚠ WARNING: This product can expose you to chemicals, including bisphenol A (BPA), which is known to the State of California to cause birth defects or other reproduction harm. For more information go to www.P65Warnings.ca.gov.

© 2021 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

Travel Rollator

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Item # RTL800KD-BK

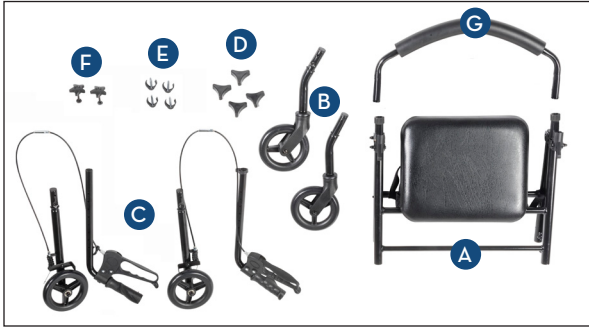
EU Authorized Representative
Y Sung Handelsvertretung
Duesselthaler Str. 24, 40211
Duesseldorf, Germany

www.drivemedical.com

assembly instructions

1. Remove all contents from packaging.

- | | |
|--|---------------------------|
| A. Frame | D. Wheel Tightening Knob |
| B. Front Wheel Assembly (Without Brakes) | E. C-Clip |
| C. Back Wheel Assembly (With Handbrakes) | F. Handle Tightening Knob |
| | G. Backrest |

2. Open frame (A) and place up-side-down on the floor. Insert the front wheel assembly (without brakes) (B) into the front left frame receptacle. Note: Correspond the front wheel assembly labeled "L" with the front frame receptacle labeled "L". Secure in place using the wheel tightening knob (D), threading the knob in from the outside. Insert c-clip (E) above the knob to secure wheel in place. Repeat for second leg, corresponding the front wheel assembly labeled "R" with the front frame receptacle labeled "R".

assembly instructions

3. Attach back wheel assembly (with handbrakes) (C) using the same process as described in Step 2 making sure the wheels are positioned on the outside of the rollator.

4. Position rollator upright and insert the left handle tube into frame and set to desired height. Secure in place using the handle tightening knob (F), threading the knob in from the outside. Note: Ensure the washer is threaded on the inside of the frame closest to the screw head. Repeat for the right handle, making sure both handles are set to the same height. Affix brake cable to the frame using the "S" shaped clamp provided.

5. Attach padded backrest (G) by depressing the black thumb buttons and sliding into the black bracket. Note that back rest can be folded up when not in use.

WEIGHT CAPACITY 350 LBS (158 KG)

safety precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from rolling. If the wheels roll while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should roll freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.

Important Safety Notice

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop—Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position—Gently press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks—Pull up on both loop locks with palms of hand to original position.

instructions to adjust the hand brake system

Important Safety Notice

The hand brake system fitted on this rollator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

Hold firmly in place
do not turn.

To tighten the hand
brake adjustment.
“turn the nut clockwise”

To loosen the hand
brake adjustment.
“turn the nut counter
clockwise”

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© 2021 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2021 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.